



BULK PACK

FLEX-STAR V Instructions for Use

Suitable for: Partial and complete dentures, occlusal splints and nightguards.


NOT Suitable for: Small unilateral restorations.

Warnings: Use eye protection and heat-resistant gloves when injecting. Use proper ventilation, vacuum system and masks when grinding prosthodontic resins.

Precautions: Keep package sealed when not in use.

This bulk material is designed for use with aluminum cartridges.

- 1) Fill cartridge with 18 to 27 grams of material.
- 2) Heat cartridge for 12 minutes.
- 3) Inject into flask.
- 4) Apply downward pressure for 3 minutes.
- 5) After removing the cartridge, let flask cool for at least 30 minutes before deflasking.
- 6) Finish using standard laboratory techniques.

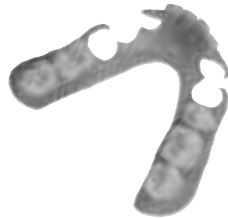

DIVISION OF CMP INDUSTRIES LLC

413 North Pearl Street • Albany, NY 12207 • USA

Toll-Free: 800.833.2343 • Outside USA: 1.518.434.3147

Fax: 518.434.1288 • www.nobilium.com